ASSEMBLY INSTRUCTIONS

Rigid Walker Frame

- NL *Looprek*
- FR *Cadre de marche*
- DE *Gehhilferahmen*
- ES *Caminador*
- PT *Andarilho*

PR30195


Aids to daily living

Able2

Please inspect all parts carefully for cracks/breaks prior to use.

STEP 1


Attach the 2 leg tubings onto the front main frame, ensure the buttons protrude completely and the 2 leg tubings are set at the same height. (picture 1)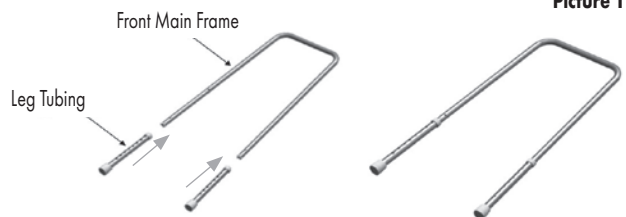

Picture 1

STEP 2

Attach the 2 leg tubings onto the rear main frame, ensure the buttons protrude completely and the 2 leg tubings are set at the same height. (picture 2)

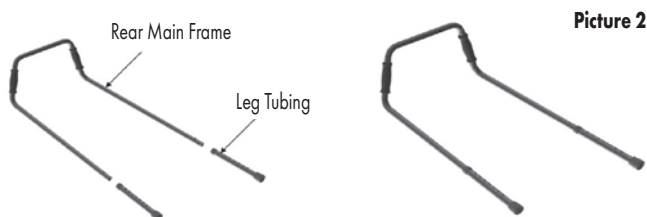Picture 2

STEP 3

Assemble the front main frame and the rear main frame using the round Philip's head screw (M5x35mm) and arch washer. Ensure the screws are tight. (picture 3)


Picture 3

Dimensions

Height adjustable from 81 – 91.5 cm
Width front: 45.7 cm
Width rear: 66 cm
Depth: 54.6 cm
Diameter aluminium frame: 25 mm


STEP 4

Set the U connection tubing to the main frame as shown on the pictures, use the M5 screw and washer to lock up the joints. (picture 4+5+6).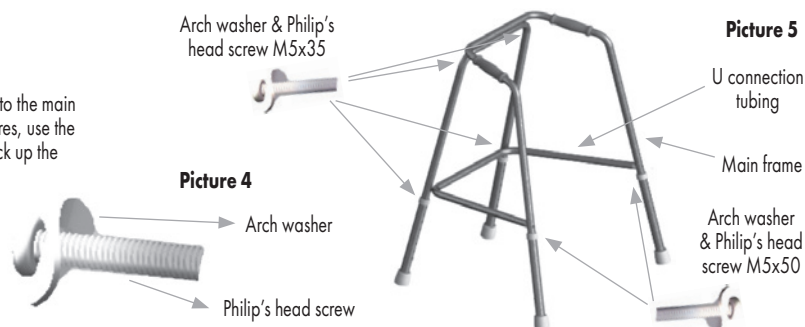

Picture 4

Picture 5

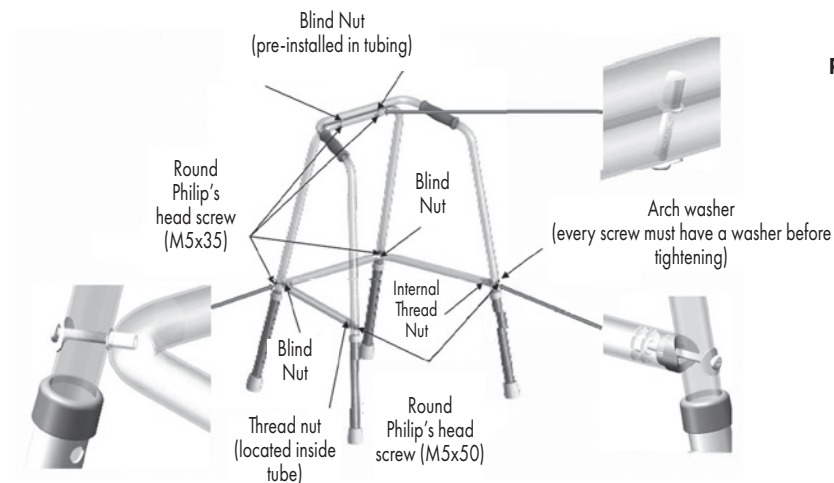Picture 6

- When adjusting the height of the leg extension, make sure that each is set into the same height adjustment hole and that the walker is level. Snap buttons must fully protrude through the adjustment hole of each leg extension to ensure a positive lock.
- Electric or pneumatic screwdriver is not suggested to use.
- Persons with severe disabilities or persons who are unable to walk without an aide should not attempt to use this product without assistance.

USER INSTRUCTIONS

This walker is designed to help you balance or unload weight from your lower extremities while you walk.

To adjust handle height: stand up straight with arms at your side. Adjust the legs so the handgrips are level with the bend of the wrist. Elbows should be slightly bend when in use, to offer control and more stability.

Make sure that the legs of the walker are level to the ground. Place the walker a comfortable distance in front of you. Using your arms for support, walk up into the frame of the walker. Do not step up completely against the front bar of the walker. This could throw you off balance. Repeat this process to continue walking.

When turning, take small steps so you won't lose your balance.

When sitting, back up all the way to the surface you are going to sit on until your legs touch the seating surface. Reach back with one hand to the seating surface to steady yourself as you sit. When rising from a seated position, push from the seating surface to rise, then place hands on the grips of the walker.

MAINTENANCE INSTRUCTIONS

Before use, check to make sure that the rubber tips are not ripped, worn, or missing. Replace if necessary before using.

CARE INSTRUCTIONS

Clean with mild cleanser and water; dry thoroughly.

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Able2 Europe B.V.
Able2 UK Ltd.
Valentine International Ltd. Taiwan