# INSTRUCTION MANUAL

## Neptune Rollator

- EN Neptune Rollator
- NL Neptune Rollator
- FR Neptune Déambulateur

**PR30265-CH**

**PR30265-BK**

*Aids to daily living*

**Able2**

## Parts List

- |                                   |                       |
|-----------------------------------|-----------------------|
| 1. Handgrip                       | 8. Front wheel        |
| 2. Adjustable tubing handgrip     | 9. Brake adjust screw |
| 3. Brake lever                    | 10. Rear wheel        |
| 4. Interlock button for handgrips | 11. Safety clasp      |
| 5. Back Rest                      | 12. Kerb climber      |
| 6. Seat                           | 13. Crutch holder     |
| 7. Bag hook                       |                       |

## Introduction

The walking aid is suitable for a single user with limited mobility. The Neptune rollator is ideal for indoor and outdoor use.

## Neptune Includes


- Backrest
- Crutch holder
- Cup holder
- Instruction manual
- Rollator bag
- 8" PU front swivel wheels and 7" PU rear wheels

## Unfolding the Rollator

Open the safety clasp and pull the operating handles apart (**see figure 1**).

Press the side bars down until you hear a small click in the fully folded position (**see figure 2**).

Before using the rollator, make sure that the folding mechanism is locked in place.

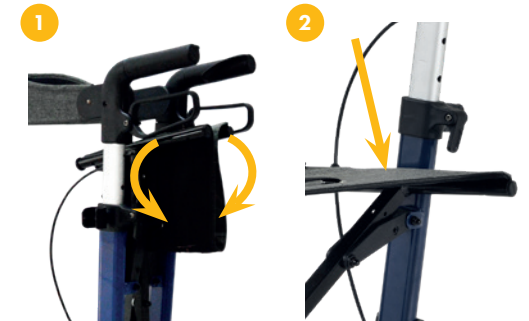## Height Adjustment

Lift the catch and push or pull the handgrip to the correct height.

Press down the catch, the handgrip will now be locked at the correct height (**see figure 3**).

## Removing / Attaching the Carry Bag

To attach the bag strap onto bag hook on the front end of the seat bar facing outward (see figure 4 & 5). The rollator bag is available for purchase.

## Braking / Locking

Pull both brake levers upwards to apply the brakes while the rollator is moving (see figure 6).

Always brake slowly and steady.

Press both brake levers down to apply the locks while the rollator is stationary (see figure 7).

Pull up the brake levers and release to release the locks. The effect of the brakes can be affected by abrasion of the wheels. Regularly test the brakes.

## Adjustment of Brakes

To adjust the braking strength, pull up the brake cable (see figure 8). Adjust the brake by pressing the nut upwards or downwards (see figure 9).

## Sitting

Position the rollator on an even and solid surface. Lock the brakes (parking position). The rollator must stand and must not slip. Sit straight up and do not lean backwards (see figure 10).

DO NOT MOVE ROLLATOR WHILE SEATED.

## Folding / Storage

To fold, pull seat upwards by the handle on the seat plate as shown in (see figure 11).

The rollator can now be folded completely and secured by using the safety clasp as shown in (see figure 12).

## Kerb Climber

We recommend you to use our built-in, kerb climber (see figure 13).

This saves you lifting the rollator over obstacles such as thresholds and kerbs. Just step down on the kerb climber and with a gentle pull back on the handles, the rollator will raise its front wheels to easily maneuver over obstacles.

## Crutch Holder

The elegant crutch holder consists of two parts, an upper hook and lower foldable base, and is used to take along a crutch or a walking stick with the rollator.

## Cup Holder

The Neptune comes with a cup holder, which is ideal for transporting drinks.

## Safety Instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 265 lbs (120 kg).
- Regularly check the screws and fasteners on the rollator and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lbs (5kg).
- Do not expose the product to temperatures above 104° F (40° C) for an extended period of time, as this can cause the rollator's parts to overheat and burn the user.
- Please make sure the rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both handgrips are fully tightened and are at equal heights. Test rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.
- **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

## Cleaning / Maintenance

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

## Specifications

Length: 65 cm  
 Width: 60 cm  
 Folded width: 22 cm  
 Adjustable height: 87 – 105 cm  
 Seat height: 60 cm  
 Weight: 7.1 kg  
 Weight capacity: 120 kg (265 lbs)

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