

Quattro

SEAT RANGE

From walking stick to restful seat in an instant!

Thank you for choosing a Linden Quattro seat. We wish you many years of happy use. For your complete safety please read all instructions:

- Quattro 32 – Walking height: 815mm (32")
Seat height: 480mm (19")
- Quattro 34 – Walking height: 865mm (34")
Seat Height: 525mm (20³/₄"
- Quattro 37 – Walking Height: 940mm (37")
Seat Height: 590mm (23¹/₄"
- Maximum user weight: 125kg

Your Quattro seat is complete with four rubber ferrules and has been designed for use on level, non slippery surfaces.

N.B. Regularly check that all rubber ferrules are in good condition. For your safety NEVER use the seat with any ferrule missing.



1. To use as a seat: Open fully and ensure all feet are in contact with the ground.
2. Sit down slowly. For maximum stability it is important when sitting to have your legs in front of you with both feet firmly on the ground.
3. The Quattro seat is intended for use only as a one person seat. Do not use it for anything else.



SPORT SEAT MODELS

The Sport seat models with the additional point and plate feet can be used on soft ground by removing all rubber ferrules. Replace all ferrules for use on hard ground.



MODELS
AVAILABLE

Polished
aluminium
frame and
coloured
canvas

or

'Deluxe'
painted frame
to match
seat colour –

COLOURS
AVAILABLE

Black or Green

