WWW.ATLANTIS-BS.EU

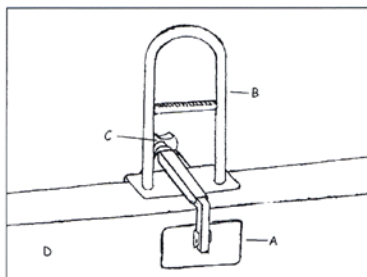
BATH TUB GRAB BAR

ATLANTIS
BATHROOMSOLUTIONS

FOR YOUR SAFETY AND COMFORT

- A Small Bracket
- B Handle
- C Tightening knob
- D Bathtub

Usage:

This product is to be used as an aid of balance and support when stepping in and out of the tub. This product is NOT designed to support the total weight of the user. When stepping in or out of the bath tub, hold the tub bar firmly and apply downward force onto the top of the bar.

DO NOT pull or push the tub bar in a sideways motion, this will loosen the tub bar from the tub and may cause injury. DO NOT use the tub bar with lipped style bath tubs. DO NOT over tighten horizontal bolts. This may cause damage to the tub bar and/or the bath tub.

Maximum user weight 100 kg

Mounting:

The bracket (A) should be on the inside of bath tub and the knob (C) on the outside of bathtub.

1. Thoroughly clean area of bath tub where tub bar is to be attached.
2. Unscrew the tightening knob (C) so that sufficient space is created to mount the bracket on the outside of the bath.
3. Lower the tub bar onto the area of bath tub (D) so that the tightening knob (C) is outside the bath tub.
4. Tighten the knob (C) so that the tub bar is securely fastened to the bath tub.

Maintenance:

Clean with mild detergent and water or antibacterial cleaner. Do not use abrasive cleaners or cloths. Dry thoroughly after each use. Periodically check for tightness. Regularly apply a silicone lubricant to screw thread of the tightening knob to prevent rust.

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Able2 Europe B.V.
Able2 UK Ltd. www.able2.uk
Ningbo Shenyu Medical Equipment Co.,Ltd.
Sungo Europe B.V.