RAISED TOILET SEAT

ATLANTIS
BATHROOMSOLUTIONS

FOR YOUR SAFETY AND COMFORT


Instructions: Attach the 2 L-shaped brackets to the raised toilet seat. Turn the model upside down on a flat surface and insert the L-shaped brackets into the slots. Push gently on the bracket and turn the knob clockwise until the thread engages. Position the brackets leaving a greater width than that of the toilet bowl.

Lift up existing toilet seat and place the seat directly onto the toilet bowl positioning the brackets towards the rear and the lip against the front of the bowl. Turn the adjustment knobs clockwise until a resistance is felt and the seat does not move from side to side. Ensure seat is firmly fixed before using.

ATTENTION: The side-locks have been especially designed to allow for easy tightening. Therefore turning should not be continued once a resistance is felt.

Care and Maintenance: All parts can be cleaned with warm water and a neutral cleanser. Alternatively, the seat can be totally immersed in water and/or can be sanitized (without brackets) in an autoclave at 85°C for 1 minute.

Caution: Do not exceed maximum user weight: 225kg. Always ensure the toilet seat is securely fixed before using. Do not hold onto seat for support while rising/lowering.

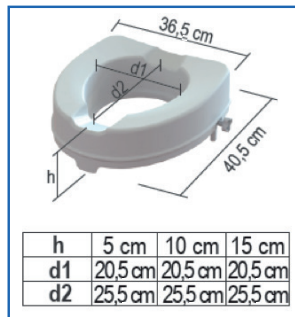Intended usage: Raised toilet seats elevate the existing toilet seat in order to ease the process of sitting down and getting up from the toilet. The models provided with lid improve hygiene conditions during toilet flushing. The raised toilet seat with lid closed can be used as a seat.

Indications and contraindications: Raised toilet seats are designed for elderly people, wheelchair users, people with reduced lumbar, knee or hip mobility and for those who have difficulty sitting down and getting up from the toilet. Independent sitting is required during use. It is not recommended to use the raised toilet seat in case of perception disorders or balance problems.

Safety warnings and indications: Strictly adhere to all assembly instructions provided in the user manual. Do not use the raised toilet seat if damaged or altered in any way. Do not bend too far forward as you sit. Do not store the raised toilet seat outdoors.

Reuse requirements: Raised toilet seats are reusable. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by a qualified dealer.

Correct disposal procedure: Raised toilet seats should not be disposed of as normal household waste. Please contact your local authorities and ask for the correct and eco-friendly method of disposal.

Incident reporting: Should an incident occur with this device please contact the manufacturer of this product at info@able2.nl and the responsible authority in your country.

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Able2 Europe B.V.
Able2 UK Ltd.
LM Global Design Limited