# Mobile Table

ES

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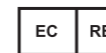
ES Mesa auxiliar para cama

PT Mesa auxiliar para cama

Valentine International Ltd. Taiwan  
8th Floor, No. 149, Sec 2, Ta Tung Road  
221 Hsichih District, New Taipei City  
Taiwan R.O.C

Y. Sung Handelsvertretung  
Duesselthaler Str. 24  
40211 Duesseldorf Germany

Distributed by:  
Able2 Europe B.V.  
Koperslagerij 15  
4762 AR Zevenbergen  
The Netherlands  
[www.able2.nl](http://www.able2.nl)

Able2 UK Ltd.  
[www.able2.eu](http://www.able2.eu)  
Able2 Iberia S.L.  
[www.able2.es](http://www.able2.es)

Aids to daily living

Able2

Aids to daily living

Able2


Please inspect all parts carefully for cracks/breaks prior to use.

#### ASSEMBLY AND USER INSTRUCTIONS

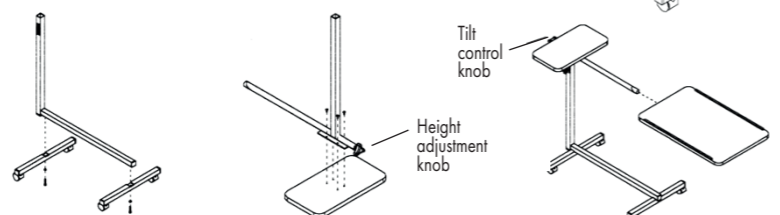The mobile table has two tops. The fixed top (20 x 38,4 x 2 cm) allows the user to keep their drink while the tilt top (60 x 38,4 x 2 cm) allows the user to turn to their desired reading angle. Both tops are made of particle board. The table top height can easily be adjusted from 70 to 110 cm. Including four casters (ø4 cm) for smooth movement. The maximum weight for the table top is 10 kg.

Do not lean on the suspended end of table top at any time. This may tip over the entire mobile table. Do not put any burning objects on the table top as it may catch fire. Make sure the tilt control knob and height control knob are tightened at all times. Otherwise items on top of the table may tip over.

1. Screw in four casters onto the two base legs.
2. Assemble two base legs to the outer mast frame.
3. Assemble the inner mast (with tilt mechanism) to the fixed top.
4. Insert the inner mast with tilt mechanism and fixed top into the outer mast frame. Then tighten the height adjustment knob.
5. Slide the tilt top into the tilt mechanism of inner mast and tighten the tilt control knob.
6. The table top height can be adjusted by releasing the height adjustment knob. The tilt top can be turned to desired angle by releasing the tilt control knob.

#### MAINTENANCE INSTRUCTIONS

Periodically check fit and tightness of all nuts, bolts, casters and adjustment knobs to make certain table is safe for use.

#### CARE INSTRUCTIONS

Gently wipe the mobile table with a damp, soft cloth to remove dust and dirt. To wash off more stubborn grime, clean with mild soap and water. Do not use cleansers that may damage the finish.

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