# Compact Triwheel Walker

## Assembly instructions

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Instructions de montage

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# Able2

Slim design for better  
manoeuvrability

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Fast & easy assembly  
No tools required

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**Please read these instructions before assembly or use of the Compact Triwheel Walker\* Please consult your retailer or health care professional for proper product selection and usage**

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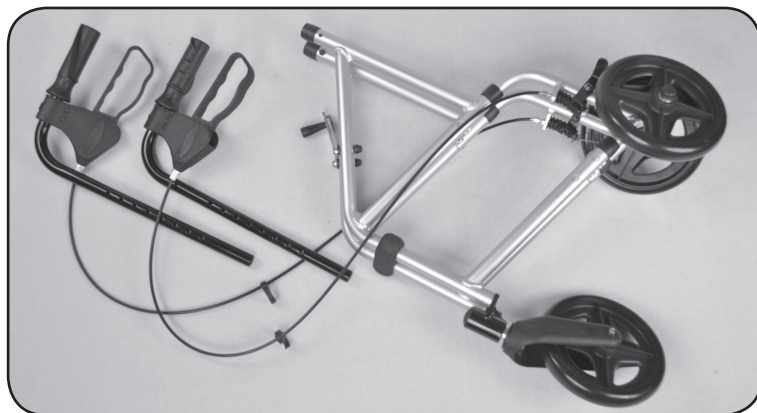
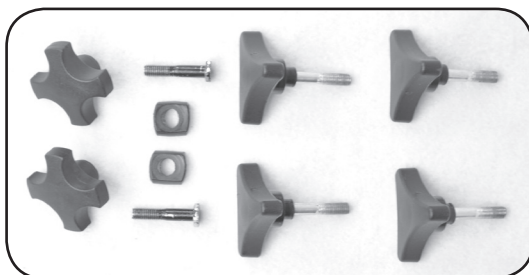
**\*Before using your new Walker please ensure that your weight is within the weight capacity of this unit.**

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**Open your Walker. Push the folding hinge until straight to secure the unit in the proper position.**

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**Pull out rear wheel until you hear a click.**

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**7** Line up holes and secure with hex-head screw, washer and hand knob. Repeat with other handle.

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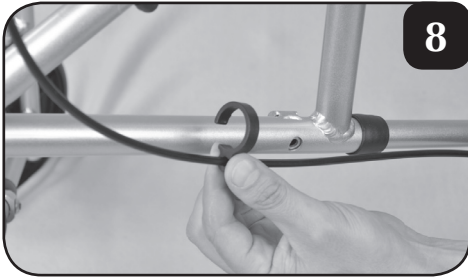
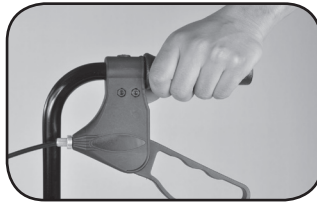
# Compact Triwheel Walker

Slim design for better maneuverability

Slank ontwerp voor betere wendbaarheid / Peu encombrant pour une meilleure maniabilité  
Schmale Konstruktion für erhöhte Beweglichkeit / Diseño ligero para mayor maniobrabilidad  
Design elegante para uma melhor maneabilidade

**This unit is designed to be used as a mobility aid, creating a natural stride as the user walks securely with their weight distributed between the handlebars with the roller walker in front of the user.**

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## Folds to fit in trunk or behind seat of car

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## Large carry bag with outside pocket and identification holder

Grote draagtas met buitenzak en naamplaatje / Grand sac avec poche extérieur et endroit pour mettre le nom et l'adresse / Großer Tragbeutel mit Außentasche und Namensschildhalter / Bolsa grande para llevar con bolsillo exterior y espacio para identificación. / Mala de transporte com um bolso exterior e suporte de identificação

# COMPACT WALKERS

## Instructions for Use, Cleaning and Care

*These instructions are guidelines only. Use only as instructed by your healthcare provider*

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### COMPACT WALKERS (PR30282/BL, PR30275/BL, PR30275/SL)

Weight capacity of compact walker is 114kg.

**WHAT ARE COMPACT WALKERS USED FOR?**  
A walker or "rollator" is a light weight frame that is used to provide walking support, with the added advantage of swivel front wheels and seat (Rollator only)

#### How Do I Adjust The Compact Walker?

1. Stand with your shoulders relaxed and your arms hanging loosely at your sides.
2. The handlebar height should be at the crease of your wrist when your arms are relaxed at your sides.
3. Loosen knobs on the outside of the handlebars, remove the bolts, then move the handlebars to the desired height.
4. Re-install the bolts and screw the knobs to the bolt. Ensure the bolt head is properly aligned with the hole before tightening fully.
5. You should be able to sit on the seat with the balls of your feet touching the ground (Rollator only).

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1. Lean on the walker, allowing it to roll forward with control as you step forward.
2. To slow the walker down (Rollator only), squeeze or pull on the handbrake.
3. Before using the seat activate the parking brake by pushing down on the handbrake.
4. To release the brake, carefully squeeze the brake handles again, being careful not to **pinch your fingers**.

**⚠ WARNING** Do not use the walker on stairs or escalators. Pay special attention on ramps or slopes.

#### COMPACT WALKER CARE AND MAINTENANCE

- Clean the walker with mild soapy water.
- Periodically check the wheels and make sure wheels spin and swivel freely, and are not excessively worn. Clean or replace if necessary.
- Periodically check the brakes to assure locking mechanism is working properly and holds when locked.
- Store walker indoors. Cover to prevent dust buildup if not in use for a long period of time.

These instructions are guidelines only.

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**Able2 UK Ltd**