
Etac Relax

Manual

etac®


785571 16-02-08

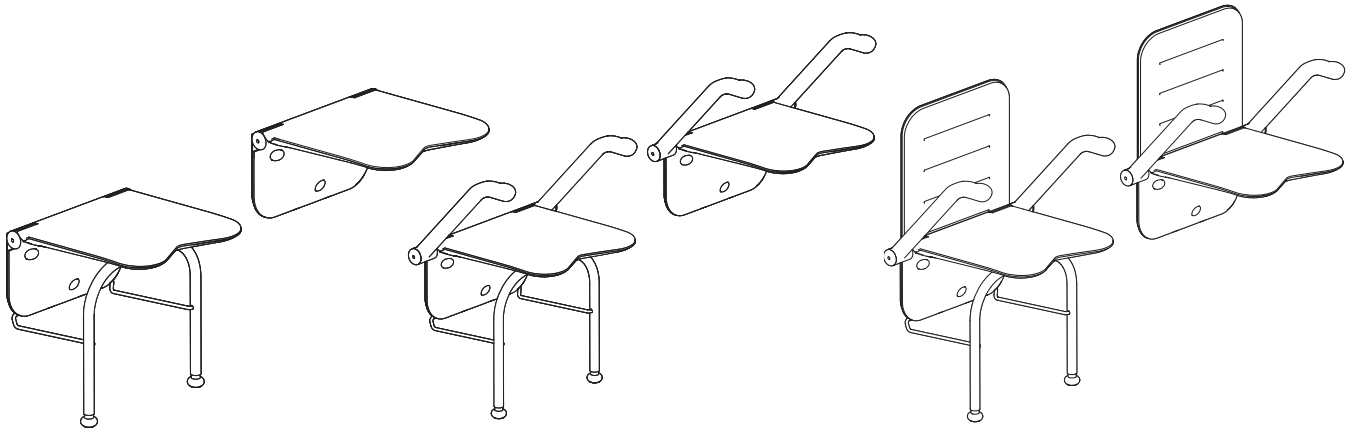
sv no da en de fi fr nl it es pt




CE

A


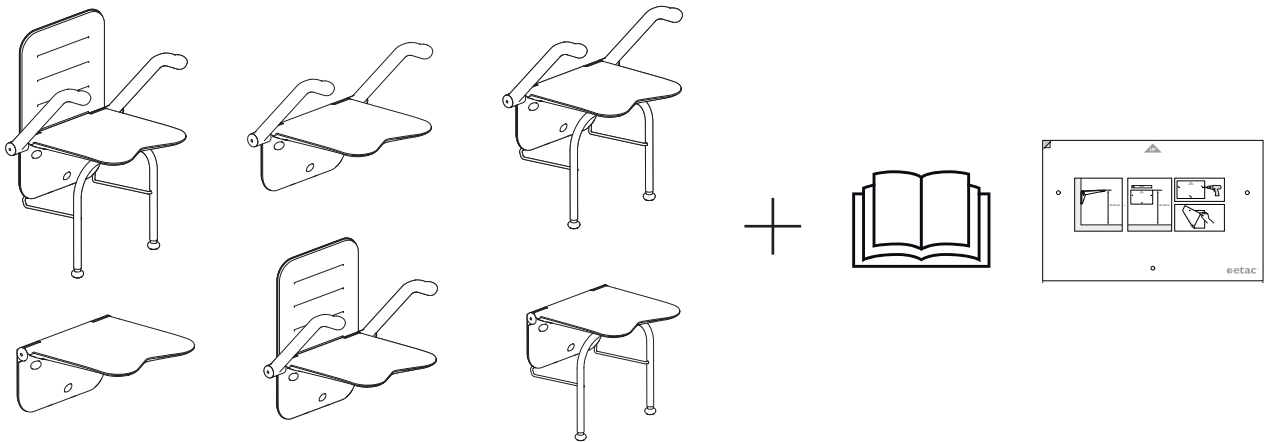


B

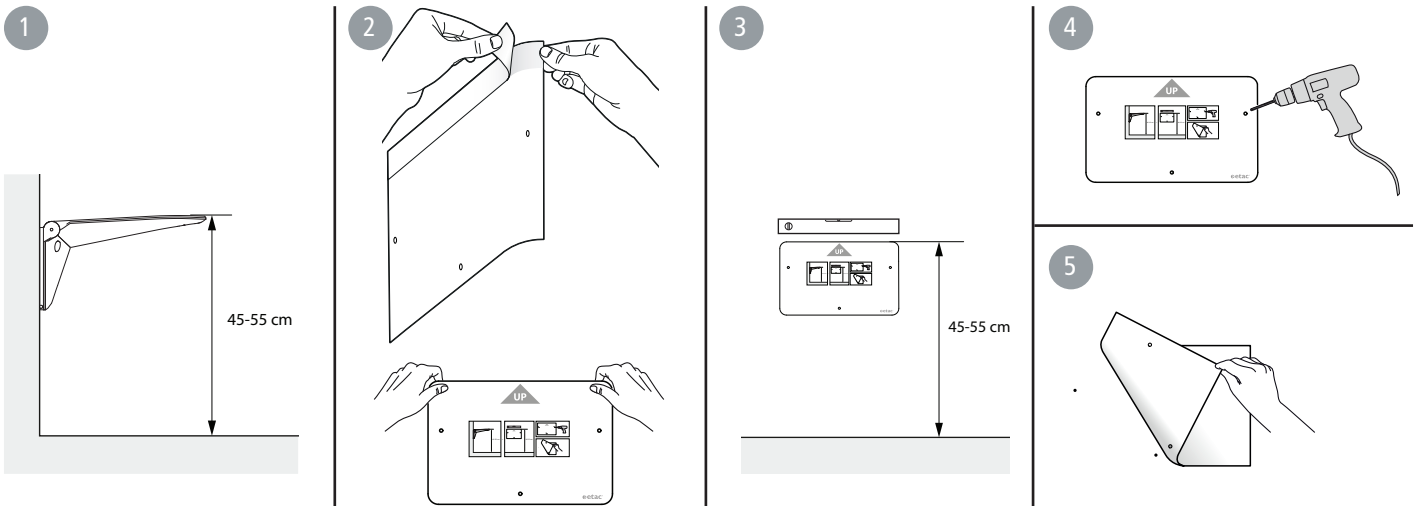
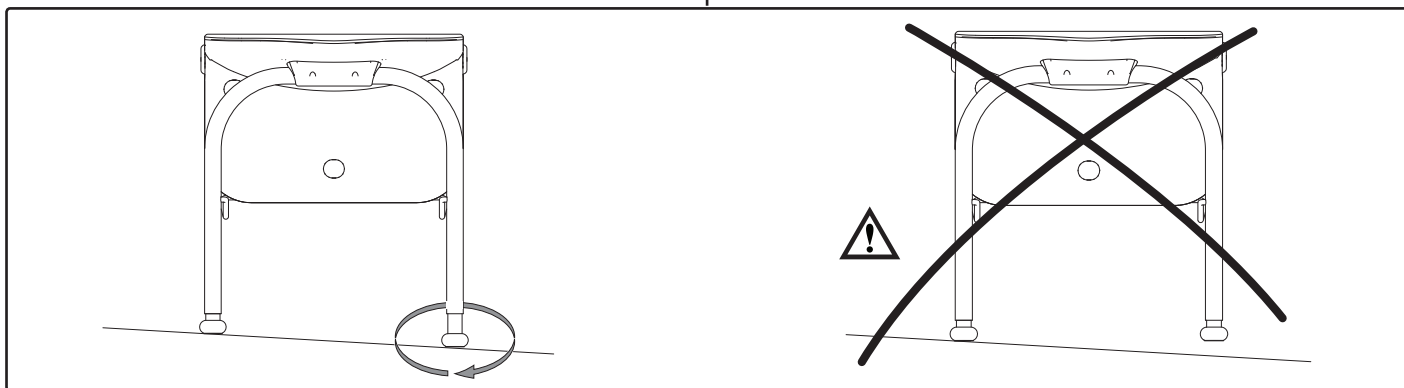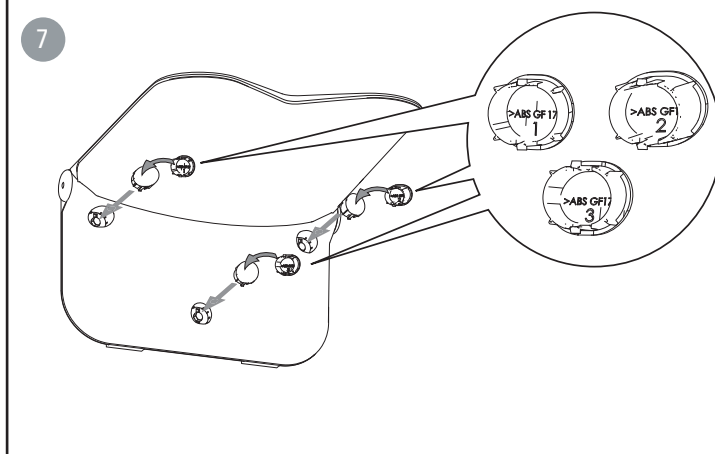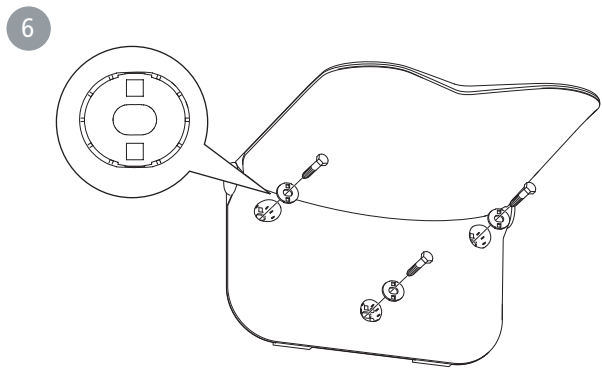
C

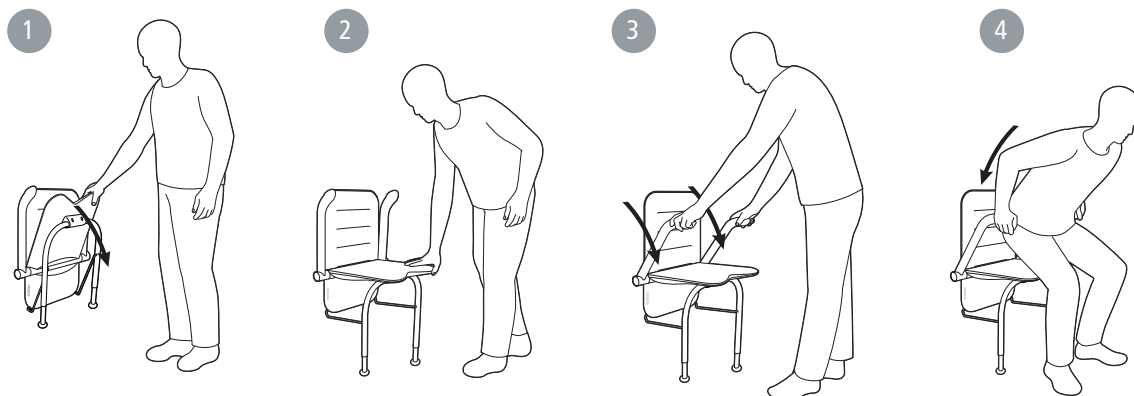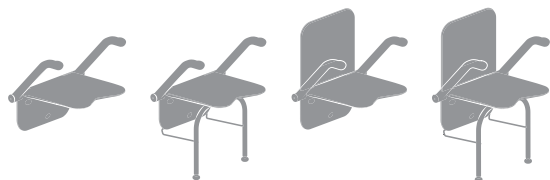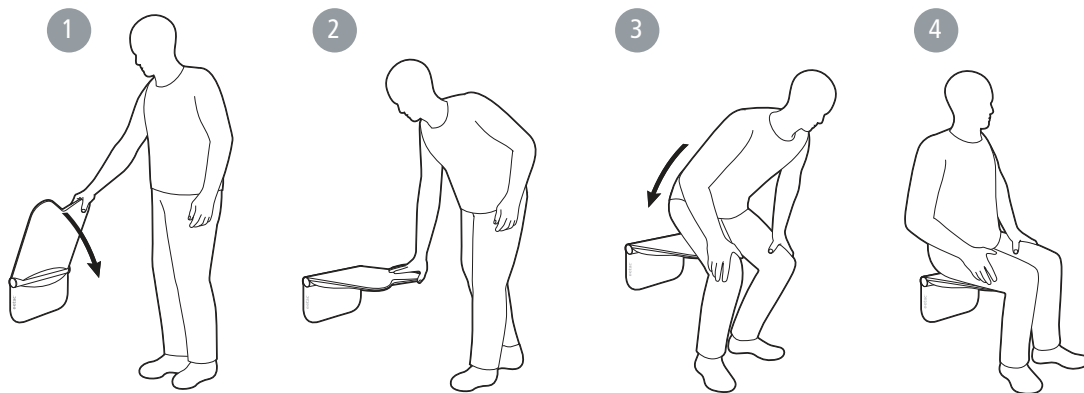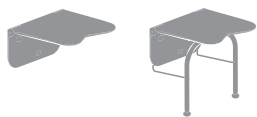
D

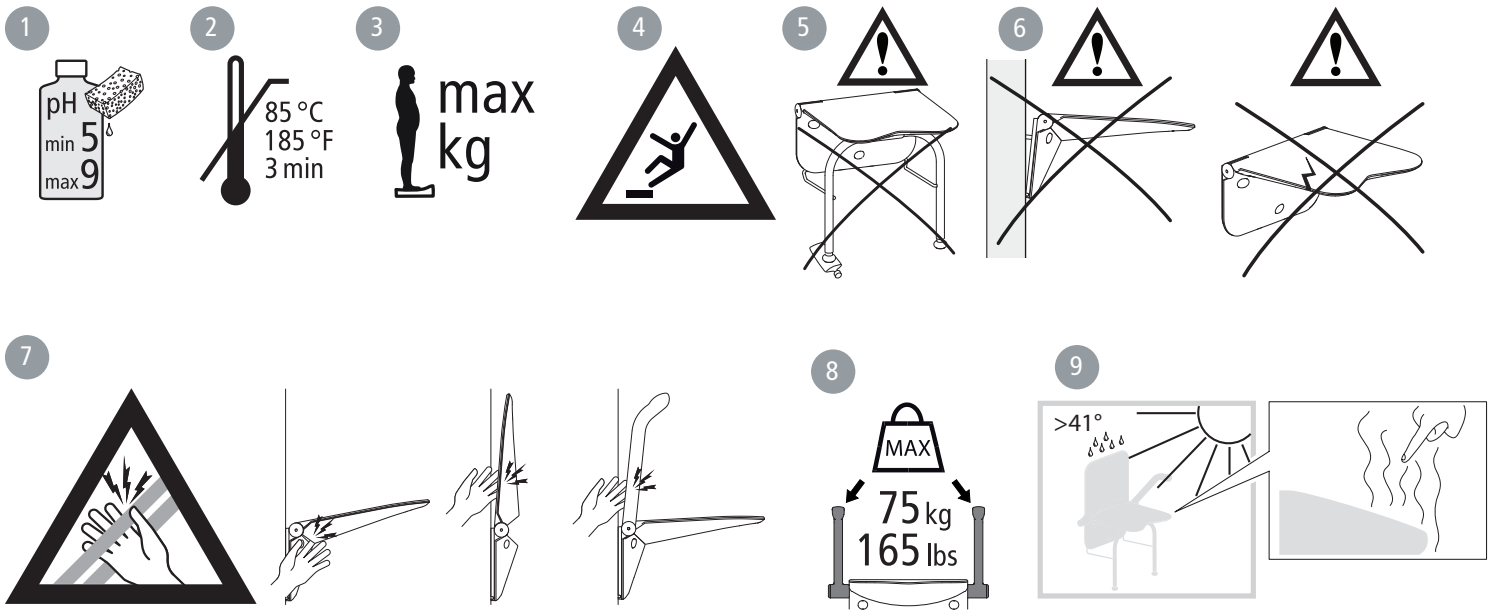E

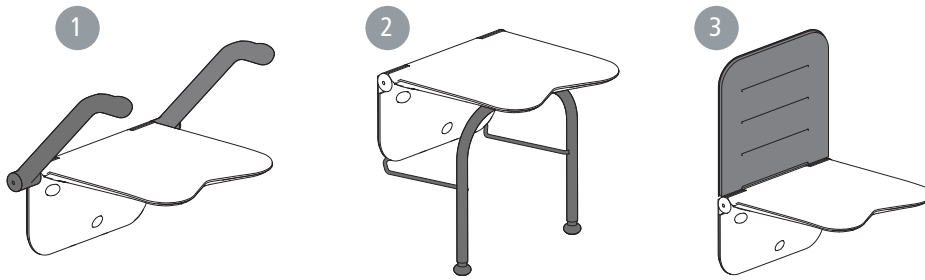F

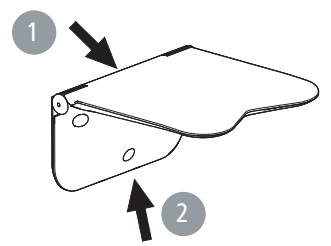G

530 mm	400 mm	390 mm	420 mm	450-550 mm	690-790 mm	450 mm					
830-930 mm	80 mm	115 mm	840-940 mm	90 mm	240 mm	120x300 mm					
MAX. 125 kg	MAX. 150 kg										
125 kg	150 kg	Alu/PPGF	Alu/TPE	Alu/TPE/PA6GF	SS	PE	3,2 kg	4,6 kg	4,0 kg	5,3 kg	0,1 kg

SV Svenska

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Tekniske data Figur G

en English

Thank you for choosing a shower chair from Etac. To avoid injury during installation, handling and use, please ensure that you read this manual thoroughly and keep it. It can also be downloaded from www.etac.com. Here you will also find other documentation such as pre-sale information, pre-purchase guide and reconditioning instruction.

Intended use

Etac Relax is a wall-mounted shower chair intended for use as a seat and standing support for personal hygiene purposes for people with impaired movement. The Etac Relax shower chair is used as a seat when there is limited space and can be folded up when not in use. The Etac Relax shower seat with supporting legs should be used when a chair is required with a higher user weight than 125 kg (max. 150 kg). The Etac Relax shower chair with armrests is recommended when support is required to stand up. The back support attaches to the wall above the chair and is recommended when back support is required. Supporting legs are recommended if the shower chair is intended for daily use by several different users.

Etac Relax is intended for use in home environments and in institutions, e.g. hospitals and nursing homes. Etac Relax is not intended for any other use or in any environment other than that stated above.

Etac Relax has been tested by testing institutes and complies with the requirements set out in EN ISO 10993-1, SS EN 12182.

Etac Relax is CE-marked in accordance with the Medical Device Directive 93/42/EEC.

Delivery contents Figur A

Installation options

May be mounted on walls with sufficient load-bearing capacity only. Please ensure that suitable fixtures are used. Each fixture must be approved for a tensile and shear load of min 4.4 kN. When mounting in non-cracked concrete C20/25 to C50/60, we recommend anchor bolt Fischer FBN II 8/30 A4 (vinyl wall covering) or Fischer FBN II 8/50 A4 (tile).

Mounting on walls with insufficient load-bearing capacity or with unsuitable fixtures can affect user safety. Mounting of the seat and assessment of the wall's load-bearing capacity must be carried out by a qualified person.

Installation Figur B

Usage Figur C

Maintenance and safety Figur D

- Clean/disinfect the chair with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry. The product is otherwise maintenance-free.
- The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.
- Maximum user weight (see technical data).
- Do not stand on the shower chair!
- Please ensure that both legs are resting against the floor when folding down the shower chair.
- Check that the bolts are securely tightened. Defective or incorrectly installed products must not be used. In case of damage, contact your dealer.
- Risk of crushing!
- The armrests have each been tested and approved for half of the maximum user weight.
- Heated chair can cause burns.

The materials used are corrosion-resistant.

The materials used are resistant to all general disinfectants.

For reconditioning and reuse, see the reconditioning instructions on www.etac.com.

The materials used are resistant to all general disinfectants.

For reconditioning and reuse, see the reconditioning instructions on www.etac.com.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

Accessories Figur E

- Arm support
- Support legs
- Back support

Label placement Figur F

- Product label (warnings, CE-marking)
- Item no label (serial no, production date)

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R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB