



LET'S TWIST AGAIN

USER MANUAL

Crutch Let's Twist Again

Field of application:

The crutch is intended as a walking aid for the walking impaired.

The product should not be used for other purposes than intended.

The product must be adjusted for the user's individual needs. Consult qualified healthcare personnel for prescription and adjustment.

Never use the tube of the crutch to stop for instance an elevator door.

If any objects, litter or pieces of paper get stuck under the ferrule, remove these immediately because it will reduce the grip against the ground.

Be extremely careful when walking on ice and other slippery surfaces.

At any sign of damage or reduced function in any respect the product must not be used.

Height adjustment is done with the button in the bottom end of the upper tube. See illustration to the right. The height must be adjusted to the individual user.

Maximum user weight 130 kgs.

Maintenance:


The crutch can be cleaned with alkaline detergent, pH 5–9.

Clean it in max 85° during max 3 minutes.

The ferrule must be inspected regularly. If it is damaged it must be replaced with a spare part. Use only original spare parts from Trustcare.

Technical specifications:

Art No. 0110 (yellow)
0111 (Orange) 0112 (Green)
0113 (Transparent).
Tubes: aluminium.
Armrest and bottom plug:
PP plastic.
Handle and ferrule:
thermoplastic rubber.
Ferrule diameter: 37 mm.
Adjustable height 80 – 100 cms
Arm section length 21 cms
(one fixed length).
Angle between cuff support line
and the leg section axis 15°.
Weight 540 g.

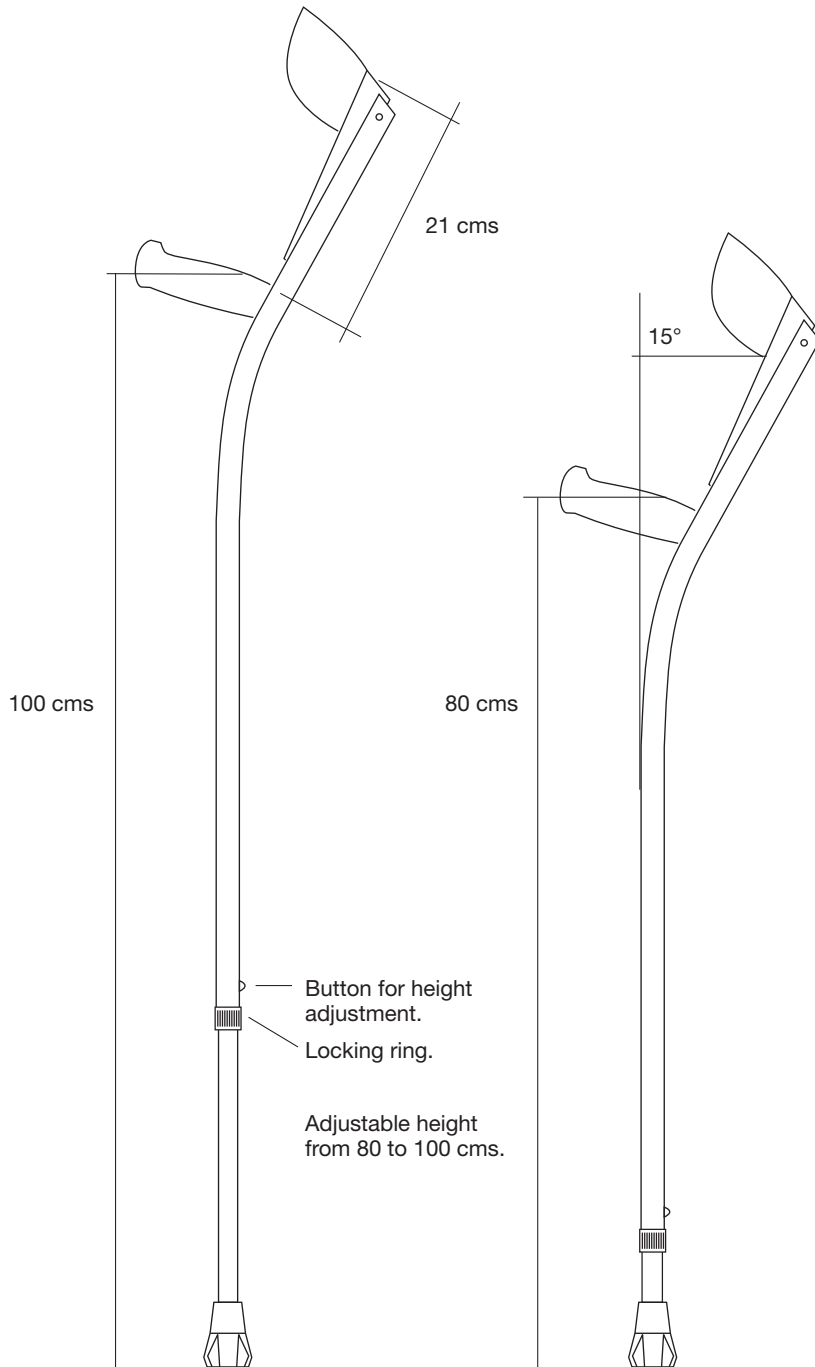



Citadellsvägen 23
SE-21118 Malmö, Sweden.
Phone +46 40 15 61 20
info@trustcare.se
www.trustcare.se