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

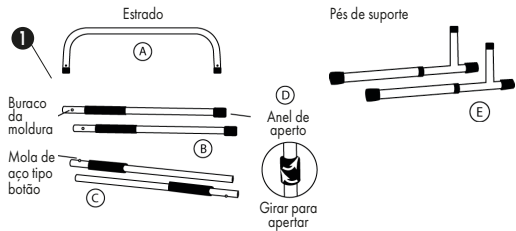
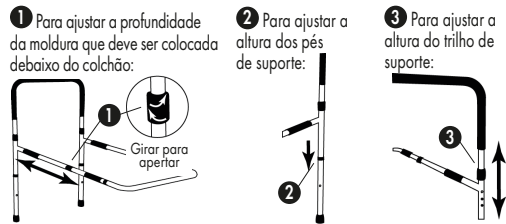
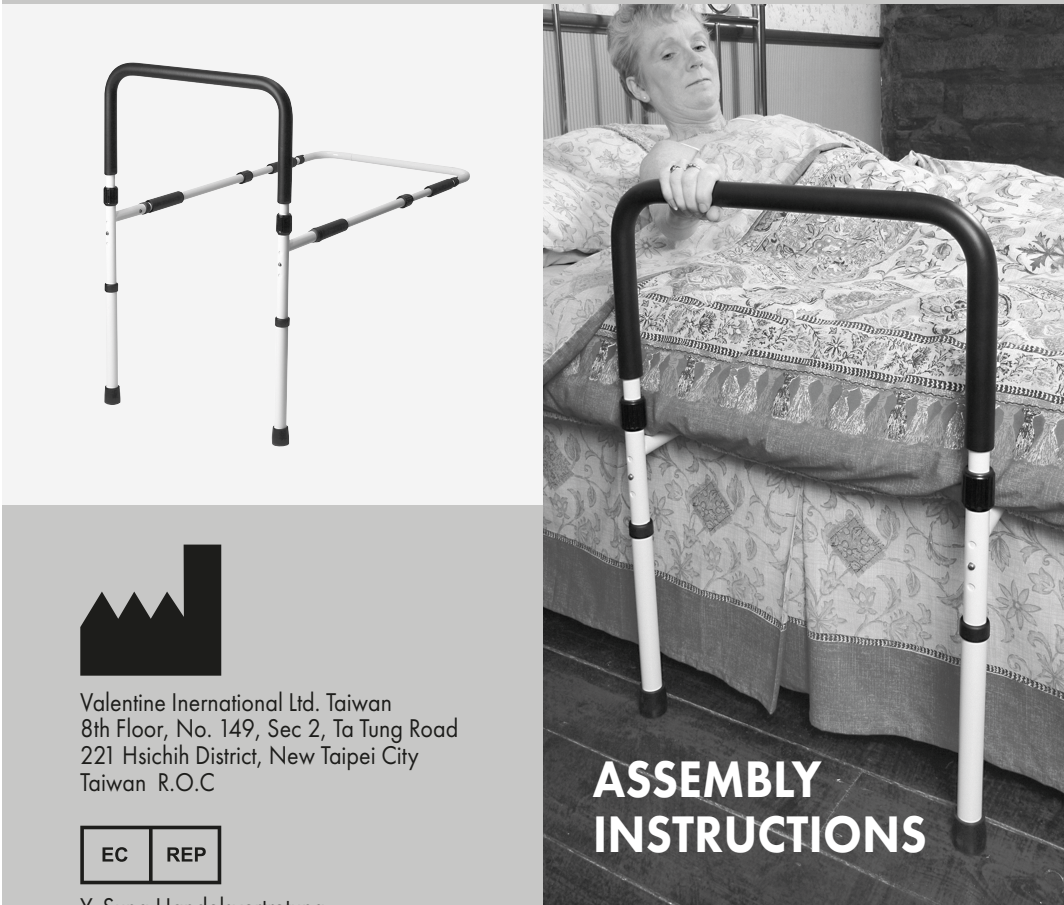
ASSEMBLY INSTRUCTIONS

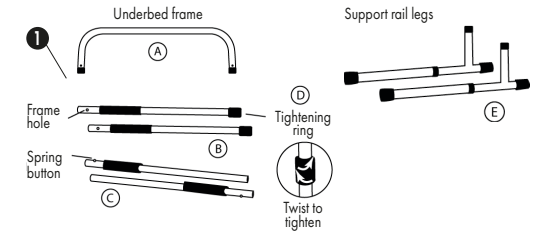GB ASSEMBLY INSTRUCTIONS

- 1 Assemble underbed frame parts (A), (B), and (C) with spring buttons. For positive assembly, be sure all spring buttons pop through the frame hole and you hear a "click". Tighten parts (B) and (C) together by twisting plastic ring (D).
- 2 Insert frames (B) into support rail legs (E).
- 3 Attach support rail (F) to underbed frame and support rail legs.

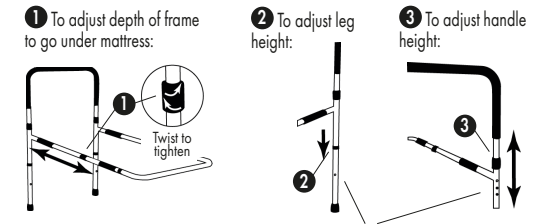ADJUSTMENT INSTRUCTIONS

Be sure push buttons are aligned equally. Push buttons must fully protrude through the adjustment hole to ensure a positive lock.

USER INSTRUCTIONS Side rails are for the purpose of aiding in turning and repositioning within the bed and are not intended to be used for restraint purposes.

This product can be used as an aid in balance and support when getting in and out of the bed. This product is NOT designed to support the total weight of the user.

CARE INSTRUCTIONS Periodically check that all push buttons are securely engaged and tightening rings are tight.

ATTENTION Make sure side rails are secured properly before use.

RISK OF SERIOUS INJURY Individuals with physical limitations who cannot prevent themselves from rolling or climbing out of the bed may require other means of safe positioning.

Side rails may be damaged if excessive pressure is placed on them.

! Please inspect all parts carefully for cracks/breaks prior to use. **!** Maximum user weight: 114 kg

Valentine International Ltd. Taiwan
8th Floor, No. 149, Sec 2, Ta Tung Road
221 Hsichih District, New Taipei City
Taiwan R.O.C

EC REP

Y. Sung Handelsvertretung
Duesselthaler Str. 24
40211 Duesseldorf Germany

CE MADE IN TAIWAN

Distributed by:
Able2 Europe B.V.
Koperslagerij 15
4762 AR Zevenbergen
The Netherlands
www.able2.nl

Able2 UK Ltd.
www.able2.eu
Able2 Iberia S.L.
www.able2.es