CONTOUR SHOWER STOOL

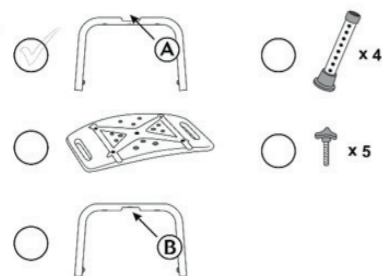
ATLANTIS
BATHROOMSOLUTIONS

FOR YOUR SAFETY AND COMFORT

Contour shower stool without backrest

Parts



Seat x 1
Leg frames x 2 (A + B)
Leg extensions x 4
35mm screws x 5

USER INSTRUCTIONS

1. Place leg frame (B) in the seat groove, and place leg frame (A) on top.
2. After both leg frames have been properly positioned place and tighten center screw, fixing both leg frames to the seat.
3. Place screws into remaining four holes to secure the leg frames to the seat.
4. Insert extension legs into leg frames. To adjust height push the adjustment button in and line up with desired hole.

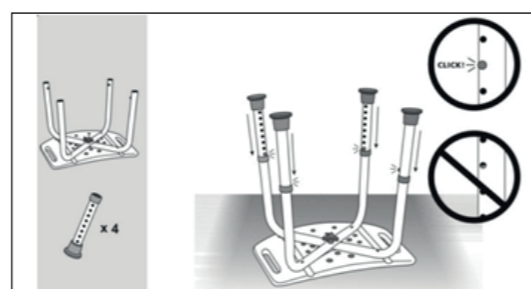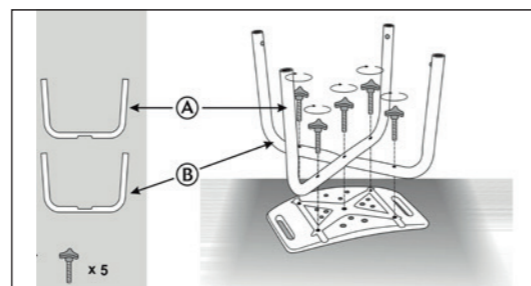
Make sure all legs are adjusted to the same height. Snap buttons must fully protrude through the adjustment hole of each leg extension to ensure a positive lock.

Contour shower stool with backrest

Parts

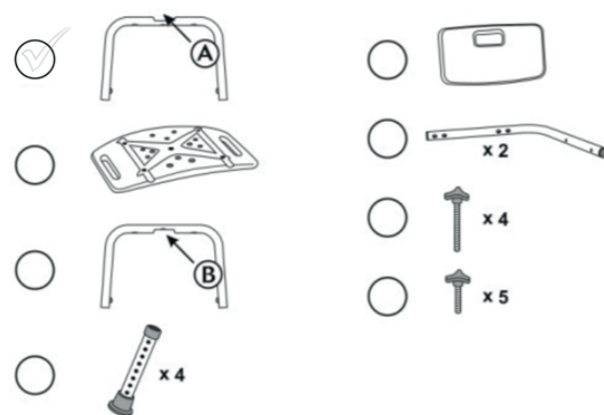Seat x 1
Backrest x 1
Leg frames x 2 (A + B)
L-Shaped Tubes x 2
Leg extensions x 4
Short (35mm) screws x 5
Long (56mm) screws x 4





USER INSTRUCTIONS

1. Place leg frame (B) in the seat groove, and place leg frame (A) on top.
2. After both leg frames have been properly positioned place and tighten center screw, fixing both leg frames to the seat. A short (35mm) screw should be used.
Do not use a long (56mm) screw to fix the leg frames to the seat, as it will damage the seat.
3. Place L-shaped tubes over the leg frames so that the holes match up. Insert four long (56mm) screws into the holes and tighten.
4. Place the seat on its side. Use four short (35mm) screws to attach the seat back to the L-shaped tubes.
5. Insert extension legs into leg frames. To adjust height push adjustment button in and line up with desired hole.

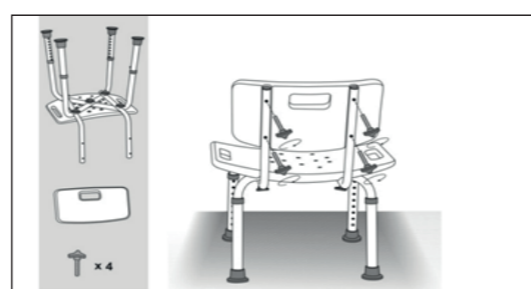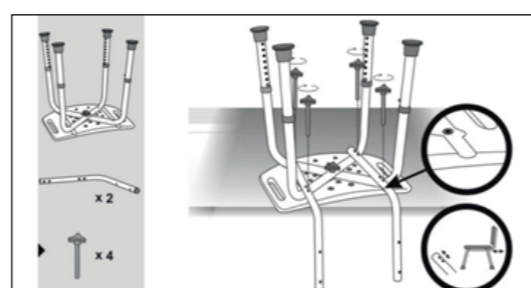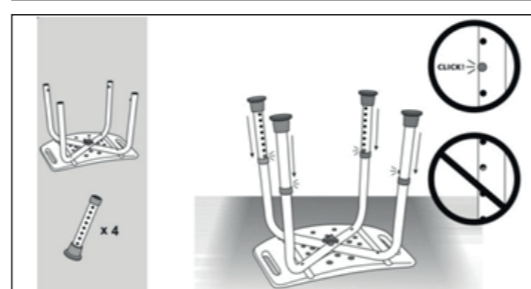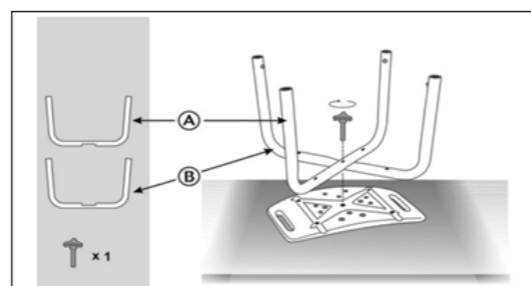Make sure all legs are adjusted to the same height. Snap buttons must fully protrude through the adjustment hole of each leg extension to ensure a positive lock.

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Able2 Europe B.V.
Able2 UK Ltd. www.able2.uk
Ningbo Shenyu Medical Equipment Co., Ltd.
Sungo Europe B.V.