EN SOFT CUSHION FOR TRIANGLE STOOL

Intended usage:
The Soft cushion is a complementary accessory for the Triangle shower stools. It makes the stool more comfortable, soft and cosy. The soft cushion covers the stool, partially protecting it from dirt and making it easier to keep it clean. The certified antibacterial properties of the composing material limit their proliferation, making the use suitable for humid and wet environments. The soft cushion can be easily applied and removed from the stool.

Indications and contraindications:
The Triangle Stool is aimed to elderly people, people with reduced mobility of lumbar, knee or hip, and to all those who have difficulty to maintain the standing position or need a further help during the use of the bathroom. The user should have autonomy of balance during the sitting and standing procedures as well as to have adequate ability to keep sitting. The triangle stool must not be used if the user has problems related to balance or perception disorders.

Safety warnings and indications:
• The maximum user weight indicated in the fitting instructions must not be exceeded.

- Before every use, be sure the stool is correctly assembled and in a stable position. Also, ensure that the soft cushion is correctly hooked to the stool by the locking clip provided and that this latter is fully engaged as shown in the pictures below.
- The shower stool can be used only on free from obstacles surfaces.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of fingers injuries may increase during mounting process.
- Do not swing on the shower stool, this may lead to slip, overturning or subsidence of the legs. Always keep the shower stool on its 3 legs.
- Do not modify the shower stool and its components in any way.
- Do not use shower stool if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- After use, the shower stool should be removed from the shower tray to avoid any obstruction.
- The Triangle Stool and its soft cushion must not be stored outdoors.

Maintenance:
The products maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth. The soft cushion must be cleaned strictly by hand.

Reuse requirements: the product is reusable. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

Correct disposal procedure: do not dispose of this product with your household waste. Contact your local authority beforehand for the correct and eco-friendly way for disposal.

Incident reporting: should an incident occur with this device please contact info@able2.uk

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